

Where to Find Battery and Pad Expiry Dates

Battery Expiration



On a Defibtech Lifeline AED, the battery expiration date is printed directly on the battery's label, usually on the back or top, often marked with an hourglass/egg timer symbol and structured as YYYY/MM/DD, indicating its readiness for use, with a typical 4, 5 or 7 year standby life for the main pack, depending on the model. The AED will alert you with a visual and audible signal when the battery needs replacing.

Pad Expiration

Defibrillation pads have a "Use Before" date clearly printed on the package label. Pads must not be used after this date, or if the packaging is torn, damaged, or already opened. The package should remain sealed until the pads are needed. Pads have a typical shelf-life of two years.

Check the pads and attached cable carefully. Make sure the connector is firmly plugged into the pad connector port, located on the corner of the AED.

Replace the pad package immediately if the pads are out of date, the packaging has been opened or damaged, or if there are any signs of cable damage such as broken wires, cuts, or nicks.

